beurer

MG 40 (Typ BF 50)

- D** Infrarot-Massagegerät
Gebrauchsanleitung
- GB** Infrared massager
Instruction for Use
- F** Appareil de massage à
infrarouge
Mode d'emploi
- E** Aparato de masaje con
infrarrojos
Instrucciones para el uso
- I** Apparocchio per massaggi
a raggi infrarossi
Istruzioni per l'uso
- TR** İnfraujlu Masaj Aleti
Kullanma Talimatı
- RUS** Прибор для массажа тела
Инструкция по применению
- PL** Aparat do masażu z promiennikiem
podczerwieni
Instrukcja obsługi
- NL** Infrarood massageapparaat
Gebruikshandleiding
- P** Aparelho de massagem de
infravermelhos
Instruções de utilização
- GR** Συσκευή μασάζ υπεράυθρων
Οδηγίες χρήσεως

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ENGLISH

Beurer electrical appliances for health and well-being meet high demands for quality, function and design.

To obtain the optimal benefits from using your infrared massager, please read the instructions for use carefully before operating the device, keep them for further reference and make them available to other users also.


Worth knowing

Massage can be recommended in the framework of all forms of health care, fitness and training. Besides increasing general well-being, it is helpful in muscle aches, muscle tension, circulatory disorders and feelings of fatigue.

The additional warmth of the infrared lamp reinforces the massage effect not only by warming the surface of the skin but also by promoting the circulation. That way the deeper layers of the tissues and muscles are also reached.

Description of device


1. Pivot for massaging head
2. Rotary knob for adjusting the angle of the handle (4 settings)
3. Slide switch
4. Infrared surface
5. Massaging surface
6. Massage fittings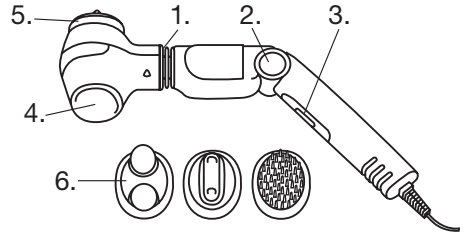

Instructions for use

Use the sliding switch to select the desired function:

Position	
0	Off
M	Vibrational massage
W	IR heat for deep results
W + M	IR heat massage

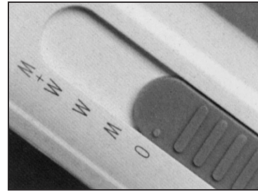
IR heat / IR heat massage


Slide the switch to 'W' to heat up the massaging surface. This then begins to light up. After a few minutes, it is heated up and ready to use.

Caution: If the massaging surface is heated up for a long time without coming into contact with the skin, the temperature could be unpleasantly high for you. Therefore, before using, test the temperature of the massaging surface with your fingertips. If the surface is too hot, it may be cooled down by briefly switching off the IR heat.

All parts of the body can be reached easily thanks to the pivoted joint of the massaging head and the possibility of adjusting the angle of the handle. The handle can be adjusted to 4 different settings by pressing the knob (Fig., no. 2). When the massaging surface is at the correct temperature, move the switch to 'W + M'. You may then start the IR heat massage.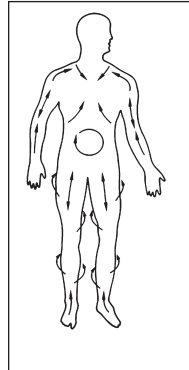

front of body

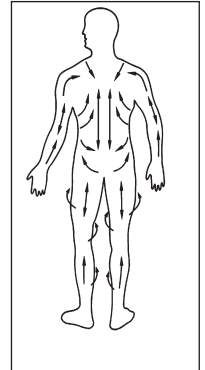
back of body

Massage

Without exerting pressure, massage using slow, circular movements. Keep the skin taut to achieve optimum results. Pictures are showing ideal massage directions.

To intensify the massage, we recommend the following massage heads:

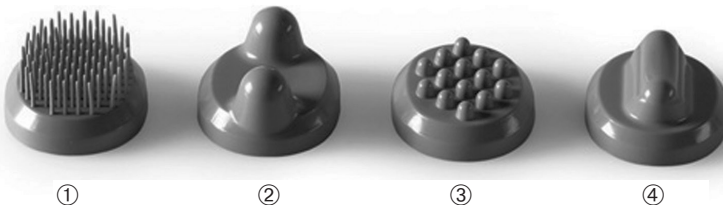- ① Brush fitting: for intensive skin massages - very pleasant, circulation - stimulating effect
- ② Knobbed fitting 2: for intensive and localised deep massages - particularly soothing after physical exertion
- ③ Knobbed fitting 1: for intensive massages to relieve muscle pain and tension
- ④ Tapered fitting: for intensive and localised deep massages - particularly soothing after physical exertion

Without the head, the massager is mainly used to massage sensitive parts of the body and to generate IR heat.

Cleaning

Pull out the plug before cleaning.

Wipe the massager with a damp cloth and then dry it. Do not use any scouring agents or solvents to clean the massager. The massage heads may be cleaned in warm, soapy water.

Only re-use the IR massager when it is completely dry.

Important Notes

- Connect the device only to the voltage stated on the type nameplate
- As with every electrical device, this massage device should be used carefully and cautiously in order to avoid the risk of electric shock:
 - unplug the mains cable immediately after use.
 - prevent any contact with water (except when cleaning with a slightly damp cloth). Water must never get inside the device. Never submerge the device in water. Never use the device in the bath, in the shower, in a swimming pool or over a basin filled with water. However, if water has got inside the casing, unplug the device immediately and contact your electrical dealer or customer service.
 - The device and in particular the mains cable should be checked for any damage before each use.
 - Never plug the device into the mains power supply if the cable is damaged. Contact your electrical dealer or customer service.
 - Repairs must be carried out only by customer service or authorised dealers.
 - Never hold or carry the device by the mains cable.
 - If a fault should occur during use, switch off the device immediately and unplug it from the socket.
- The device is intended only for the purpose described in these instructions for use. The manufacturer can accept no liability for injury arising from incorrect or careless use.
- Do not use any massage attachments other than those supplied with the device.
- The device is only intended for individual use, and not for medical or commercial use.
- Do not use the device continuously for longer than 15 minutes to avoid overheating. Let the device cool down before using it again.
- The surface of the device becomes warm during use. Persons insensitive to heat should be particularly careful.
- Never cover the device when it is switched on (with blanket, pillow etc.)
- Never operate the device unsupervised, especially if there are children in the vicinity.
- Children, helpless or disabled persons must use the device only under adult supervision or must be familiarised with the device beforehand.
- Do not use with animals or pets.
- Persons with cardiac pacemakers are requested to consult their doctor before using the massage device and never to massage the region of the heart.
- Never use the device on swollen, burned or injured areas of skin. The following should also not be massaged: bones (e.g. joints, spine), the head or other sensitive areas of the body.
- Do not carry out a massage if you have a cold with a high temperature, varicose veins, thrombosis, vein inflammation, jaundice, diabetes, pregnancy, nerve disorders (e.g. sciatica) or acute inflammation. In such cases, consult your doctor.
- Do not use the massage device in bed. You must never fall asleep during use.
- Keep the cord away from heated surfaces.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.

Please dispose of the device in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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