<b>DE</b>	Fußsprudelbad Gebrauchsanweisung.....	2	<b>IT</b>	Idromassaggio plantare Istruzioni per l'uso.....	31
<b>EN</b>	Foot Spa Instruction for Use.....	10	<b>TR</b>	Ayak jakuzisi Kullanma Talimatı .....	38
<b>FR</b>	Bain bouillonnant pour les pieds Mode d'emploi .....	17	<b>RU</b>	Гидромассажная ванна для ног Инструкция по применению .....	45
<b>ES</b>	Hidromasaje para pies Instrucciones para el uso .....	24	<b>PL</b>	Wanienka do hydromasażu stóp- Instrukcja obsługi .....	52

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

## **WARNING**

- The unit is only intended for domestic/private use, not for commercial use.
- This device may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely, and are fully aware of the consequent risks of use.
- Children must not play with the device.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- If water leaks from the appliance, the appliance should no longer be used.
- The device has a hot surface. Persons insensitive to heat must be careful when using the device.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to adhere to this instruction will void the warranty.

## Contents

1. Items included in the package.....	10	7. Use .....	14
2. Explanation of symbols.....	11	8. Cleaning and storage .....	15
3. Warnings and safety notes .....	11	9. Disposal.....	15
4. Proper use.....	13	10. Technical data .....	15
5. Description of the appliance.....	13	11. Warranty/service.....	15
6. Initial use .....	14		

## 1. Items included in the package

- 1 x Foot spa
- 1 x These operating instructions
- 1 x Massage attachment
- 1 x Brush
- 1 x Portable pedicure device

## 2. Explanation of symbols

The following symbols are used in the operating instructions and on the type plate:

	<b>WARNING</b> Warning of risks of injury or health hazards.
	<b>CAUTION</b> Safety information about possible damage to appliance/accessories.
	<b>Note</b> Important information.
	The device has double protective insulation and therefore complies with protection class 2.
	Only use indoors
	Read the instructions for use
	Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).
	Manufacturer
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.
	United Kingdom Conformity Assessed Mark
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.
	Dispose of packaging in an environmentally friendly manner
<b>IPX0</b>	Not protected against water

## 3. Warnings and safety notes

Failure to observe the information provided below can lead to personal injury or material damage. Always pass on these instructions together with the appliance if you sell it or pass it on.

### Note

- Keep children away from the packaging materials. Risk of suffocation.
- Before use, make sure that the device and accessories do not show any visible signs of damage. If you are in any doubt, do not use it and contact your dealer or a specified customer service address.
- If any fault or malfunction should occur during use, switch the unit off immediately and remove the plug from the mains socket. Never attempt to repair the unit yourself. Improper repairs can expose the user to considerable risks.
- Do not stretch, twist or kink the power cord. Do not stick in any pins, needles or sharp objects.
- Do not position the power cord on or pull it over sharp or pointed objects.
- Insert the plug into a socket that is protected from moisture, water and water spray.
- Children must be supervised when using this device.
- To fill the foot spa, it must not be placed in the shower, bath tub or wash basin.

- Do not open the case under any circumstances.
- If the device has been dropped or otherwise damaged, it must not be used again.
- Repairs to electrical appliances must only be carried out by qualified persons. Incorrect repairs can lead to considerable danger for the user. In the event of repairs, please contact our customer service or an authorised dealer.
- Cleaning and user maintenance must not be performed by children unless supervised.

## Electric shock

### WARNING

Just like any other electrical appliance, this foot spa must be handled with due care and attention in order to prevent the risk of electric shock.

For this reason, the device must only be operated as follows:

- With the mains voltage specified on the device only.
- Do not use the device if damage to the device itself or any of its accessories is evident.
- Never use the appliance during a thunderstorm.

In the event of a defect or malfunction, switch the device off immediately and disconnect it from the power supply. Never pull on the power cable or on the appliance in order to disconnect the plug from the mains outlet. Never hold or carry the device by the power cable. Keep the cables away from hot surfaces.

You must never connect the foot spa to the power supply when your feet are already in the water. If the device is damaged, this can lead to life-threatening electric shock.

Make sure that the plug and the power cord do not come into contact with water or other liquids.

For this reason, the device must be operated as follows:

- only in dry inside rooms (e.g. never in the bathtub or sauna),
- only with dry hands.

Never immerse the device in water. Never try to retrieve the device if it falls into water. Disconnect the plug immediately from the mains outlet.

## Fire hazard

### WARNING

There is a risk of fire if the device is used incorrectly or if it is not used according to these instructions for use.

For this reason, the massage device must only be operated as follows:

- Never leave the device unsupervised, particularly if children are around.
- Never use the device under covers, such as blankets or pillows.
- Never use the device in the vicinity of petrol or other easily flammable materials.
- Do not place on soft carpeting because the vents may be covered.

## Handling

### CAUTION

- Switch the device off and disconnect the power supply after every use and before every cleaning.
- Keep the device away from high temperatures.
- Never stand in the foot spa.
- The device must not be connected to the power supply while being filled.
- Always switch the foot spa off and remove the plug from the mains socket before moving it or cleaning it.
- Be aware that some spa products may create a great deal of foam when combined with the bubble massage. For this reason, do not leave the foot spa unsupervised and switch it off immediately if excess foam builds up.
- Warm foot spas with arnica, rosemary or sea salt extracts get the circulation going. Lukewarm foot spas with lavender or thyme have a relaxing effect and are a blessing for exhausted feet.

## 4. Proper use

This device is solely intended for the care of your feet.

### WARNING

Do not use the foot spa

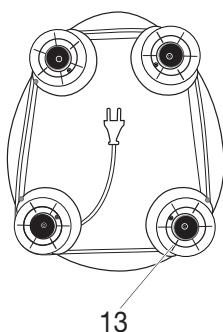
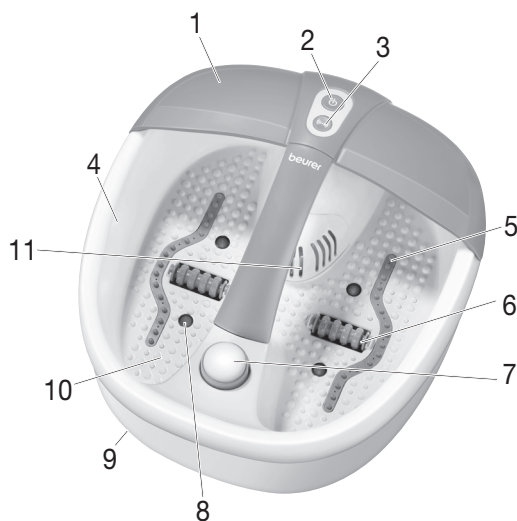
- if there are changes caused by disease or injury in the region of your foot (e.g. open wounds, warts, fungal infection),
- with animals,
- for longer than 40 minutes (danger of overheating) and allow it to cool for at least 15 minutes before using it again.

You should consult your doctor before using the foot spa

- if you are not sure whether the foot spa is suitable for you,
- if you suffer from a serious illness or are recovering from an operation on one of your feet,
- if you have diabetes or thrombosis,
- if you have leg or foot conditions (e.g. varicose veins, vein inflammations),
- if you have pains of unclear origin.

The device is only intended for the purpose described in these instructions for use. The manufacturer cannot be held liable for damage occurring as a result of inappropriate or careless use.

## 5. Description of the appliance

1. Splash protection
2. Function button
3. Function button
4. Fill-level mark
5. Bubble bar
6. Massage roller attachment
7. Interchangeable pedicure attachments
  - a Callus remover
  - b Brush
  - c Massage attachment
8. Infrared light dots
9. Rubber feet (on underside of device)
10. Foot rest with massage nodules
11. Filter cover
12. Removable aroma filter
13. Cord winder (on underside of device)

## 6. Initial use

- Remove the packaging
- Check the device, power supply and cables for damage.
- Before you plug in the foot spa, fill it up to the fill-level mark (approx 2-3 cm full) with either warm or cold water. The foot rest should, however, be completely covered with water.
- Route the cable safely so that there is no risk of anyone tripping over it.
- Place the foot spa on a firm and level surface.

## 7. Use

Make yourself comfortable in a chair and place your feet on the foot rest and apply slight pressure. Never stand up in the foot spa. The rubber feet on the base of the unit prevent it from sliding.

The device has two massage functions:

- Function 1 : bubble massage, infrared light and water temperature regulator
- Function 2 : activation of vibration massage

### Bubble massage, infrared light and water temperature regulator

- Press the  - button to switch on the bubble massage (with infrared light and water temperature regulator).
- Press the  - button to switch the bubble massage (with infrared light and water temperature regulator) off again.

These functions are only available together.

### Optional vibration massage

While the bubble massage with infrared light and water temperature regulator is running, you have the option to switch on the vibration massage. The vibration massage will only function when the bubble massage with infrared light and water temperature regulator is switched on. It cannot be used separately.

- Press the  - button to activate the vibration massage.
- Press the  - button to switch the vibration massage off again.

### Massage roller

You can experience a special massage effect by moving your feet back and forth on the 2 massage rollers integrated into the bottom of the foot spa. The massage rollers can optionally be removed by pulling them firmly upwards. When installing the rollers, make sure that the roller fixing points downwards, as shown in the illustration.

### Removable attachments

Place the chosen attachment onto the fixture provided. The **brush attachment** can be used to stimulate the reflex zones in the soles of your feet. The **massage attachment** provides a soothing massage for your feet and promotes the circulation. The **corn and callus remover** is used to remove dead skin cells and calluses.

### Magnetic field and infrared light

The foot spa features 16 magnets and 4 infrared light dots which are integrated into the foot tub. They help to boost circulation and stimulate reflexology points.

### Removable aroma filter

The removable aroma filter allows you to easily add bath additives such as bath crystals, bath salts etc. To do this, remove the filter cover [11] from the device by pulling it upwards. Remove the aroma filter [12]. Add bath products up to the level of the mark on the inside of the filter/underneath the first line of holes. Now attach the filter underneath the filter cover. When doing so, note the arrow marks on the filter and foot spa; these indicate the correct position of the filter. The two arrows must point to each other. Reattach the filter cover. The filter can be easily removed and cleaned after use. Rinse the filter with clean water. Please observe the fill-level mark on the filter and do not fill the filter above this mark. Be aware that some spa products may create a great deal of foam when combined with the bubble massage. For this reason, do not leave the foot spa unsupervised and switch it off immediately if excess foam builds up.

### **Note**

- Massaging of a specific point on the sole of the foot may influence the corresponding organ of the body. However, we recommend studying relevant literature and asking your doctor before using point reflex zone massage to achieve the desired effect.

After use, pour the water out over the narrow side of the foot spa.

## 8. Cleaning and storage

After use, rinse the unit with tap water. The foot spa can be cleaned with a mild, non-foaming household cleaner, e.g. neutral vinegar.

To dry out the air line after cleaning, switch on the bubble setting for approximately 10 seconds. Wipe away any drops that may leak with a cloth.

### **Note**

- Do not allow water to penetrate the interior of the foot spa.
- Store the foot spa in its original packaging in a dry area.
- Do not leave it open for an extended period.

## 9. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 10. Technical data

Supply:	220-240 V~, 50/60 Hz, 140 W
Dimensions:	40.0 x 37.0 x 20.6 cm
Weight:	approx. 1920 g

## 11. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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