<p><b>(D)</b> <b>Massagesessel</b> Gebrauchsanleitung</p>	2-14	<p><b>(I)</b> <b>Poltrona per massaggi</b> Istruzioni per l'uso</p>	54-66
<p><b>(GB)</b> <b>Massage chair</b> Instructions for use</p>	15-27	<p><b>(TR)</b> <b>Masaj koltuğu</b> Kullanım kılavuzu</p>	67-79
<p><b>(F)</b> <b>Fauteuil de massage</b> Mode d'emploi</p>	28-40	<p><b>(RUS)</b> <b>Массажное кресло</b> Инструкция по применению</p>	80-92
<p><b>(E)</b> <b>Sillón de masaje</b> Instrucciones de uso</p>	41-53	<p><b>(PL)</b> <b>Fotel do masażu</b> Instrukcja obsługi</p>	93-105

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**Dear customer,**

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of beauty, heat, gentle therapy, blood pressure/diagnosis, weight, massage and air.

Please read these instructions carefully and observe the information they contain.

With kind regards,  
Your Beurer Team

1. Intended use.....	15
2. Safety notes.....	15
3. Getting to know your device .....	17
4. Unit description .....	18
5. Initial use.....	23
6. Operation.....	25
7. Maintenance and storage .....	26
8. What if there are problems?.....	27
9. Technical Data.....	27
10. Disposal.....	27
11. Warranty and service .....	27

**Signs and symbols**

The following symbols appear in the instructions for use and on the type plate:

**Warning!**

Warning instruction indicating a risk of injury or damage to your health.

**Caution!**

Safety note indicating possible damage to the device/accessory.

**Important!**

Note indicating important information.

Only use in closed rooms.

Do not insert needles!

Cross-reference to another section.

**Included in delivery**

- Massage chair
- Seat cushion
- Back cushion
- Pillow
- Hand switch

**1. Intended use**

The massage chair is intended for the purpose of massaging the back, legs and feet.

Please observe the safety notes in these instructions for use.

Use the device only according to the methods specified.

The device is suitable only for **private use** and not for medical or commercial purposes.

**2. Safety notes**

- Read these instructions for use carefully. Non-observance of the following information may result in personal injury or material damage.
- Store these instructions for use in a safe place and make them accessible to all users.
- Make sure you include these instructions for use when handing over the device to third parties.

**Electric shock**

 **Warning!**

- As with every electrical device, use this massage chair carefully and cautiously in order to avoid hazards resulting from electric shock.
- Use the device only with the mains voltage specified on the device.
- Never use the device if it or any of the accessories show visible damage.
- Do not use the device if it was accidentally splashed with water.
- Do not use the device during a storm.

- Only use in closed rooms and avoid direct sunlight.
- In the event of defects or operational faults, switch off the device immediately. Pull the mains plug out of the socket.
- Never pull on the mains cable to remove the mains plug from the socket.
- Do not clamp or pinch the mains cable.
- Keep the mains cable away from hot surfaces.
- Ensure that the device, switch and cable do not come into contact with water, steam or other liquids.
- Use/operate the device only with dry hands.
- Do not subject the device to any impacts or drop the device.

## Repairs

### **Warning!**

- Do not open the device.
- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Service or an authorised retailer. Failure to do so invalidates the warranty.
- If the mains plug or mains cable of the device are damaged, they must be replaced by the manufacturer or at an authorised customer service point.
- The zip fastening of the massage chair must not be opened. It is only used for technical production reasons.
- Do not insert needles or other sharp objects into the chair.

## Risk of fire

### **Warning!**

There may be a risk of fire if the device is used improperly or these instructions for use are ignored!

- Ensure that the vents are free at all times. Always keep the vents free from dust, hair etc.
- Do not use the device together with other electrical devices (e.g. electric blanket).
- Never use the device near petrol or other highly flammable substances.

## Operation

### **Warning!**

This device is exclusively intended to massage the back, legs and feet of humans. It cannot replace medical treatment. Do not use the massager if one or several of the following warnings applies to you.

#### **Do not use the massager**

- If you suffer from a medical abnormality or injury to your back (e.g. slipped disc)
- On children
- During pregnancy
- While sleeping
- On animals
- Whilst undertaking any activity where an unexpected reaction could be dangerous
- After the consumption of substances that lead to limited cognitive ability (e.g. pain medication, alcohol)

If you are unsure whether the massager is suitable for you, please consult your doctor.

#### **You should consult your doctor before using the massager**

- If you suffer from a severe illness or recently had surgery on your upper body
- If you have a pacemaker, implant or other aids
- Thromboses
- Diabetes
- If you are suffering pain and the cause of which is unknown
- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or are instructed by this person on how to use the device.
- Do not touch the massage mechanism or the rollers with your hands during operation; there is a danger of crushing.
- Do not insert any objects in the device apertures or in the rotating parts. Ensure that the moving parts can move freely at all times.
- Children should be supervised around the device to ensure they do not play with it.
- Always switch off the device at the main switch after use.

- Ensure that the mains cable does not pose a trip hazard.
- Never stand on the device or place any objects on the device.
- Never massage areas that exhibit swellings, infections or eczema.
- Do not use the device in the event of inexplicable stomach ache.
- Only one person may use the massage chair at any given time.

- Never load the device with more than 200 kg.

 **Caution!**

- Do not expose the device to direct sunlight.
- Do not use the device at temperatures above 40°C.
- Check the function of the device if it has not been used for a longer period of time.



### 3. Getting to know your device

You can enjoy your own massage at home with the Beurer MC5000 massage chair. The massage chair offers different types of massage to give you a soothing and revitalising massage.

There are three massage programmes that focus on different areas. In addition, the massage chair also features an air pressure massage in the seat, calf and foot area.

## 4. Unit description

### External features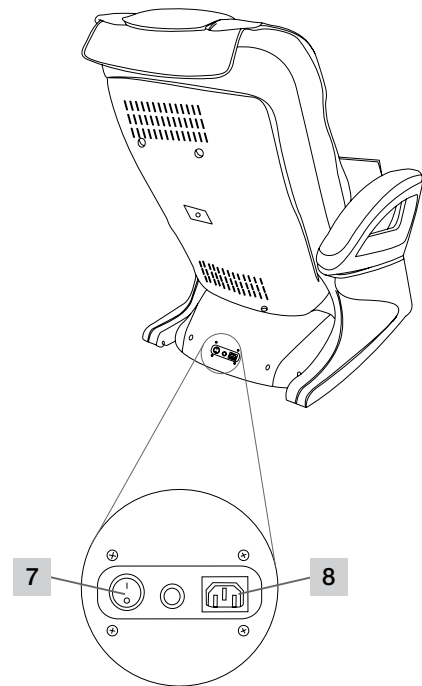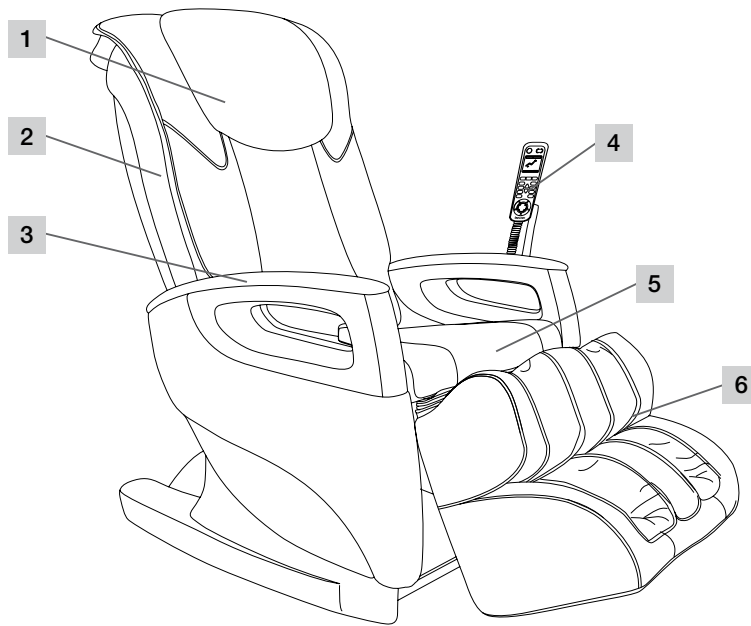

1 Pillow

2 Back rest

3 Arm rest

4 Hand switch


5 Seat cushion

6 Leg rest

7 Main switch

8 Mains socket


### Internal features

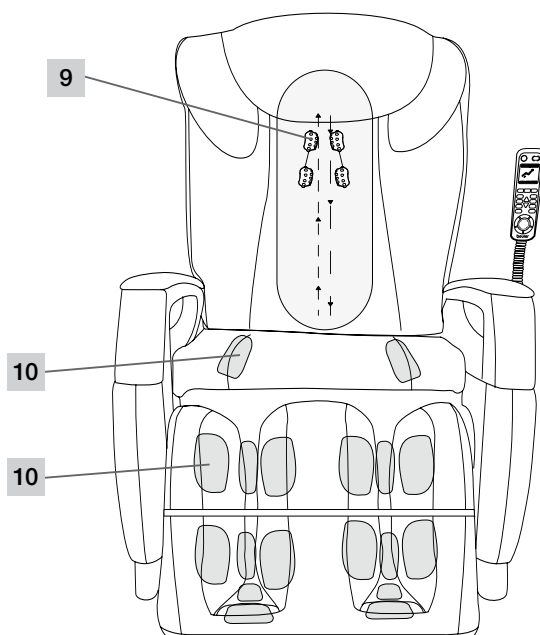9 Massage rollers

10 Air pressure massage points

## Hand switch

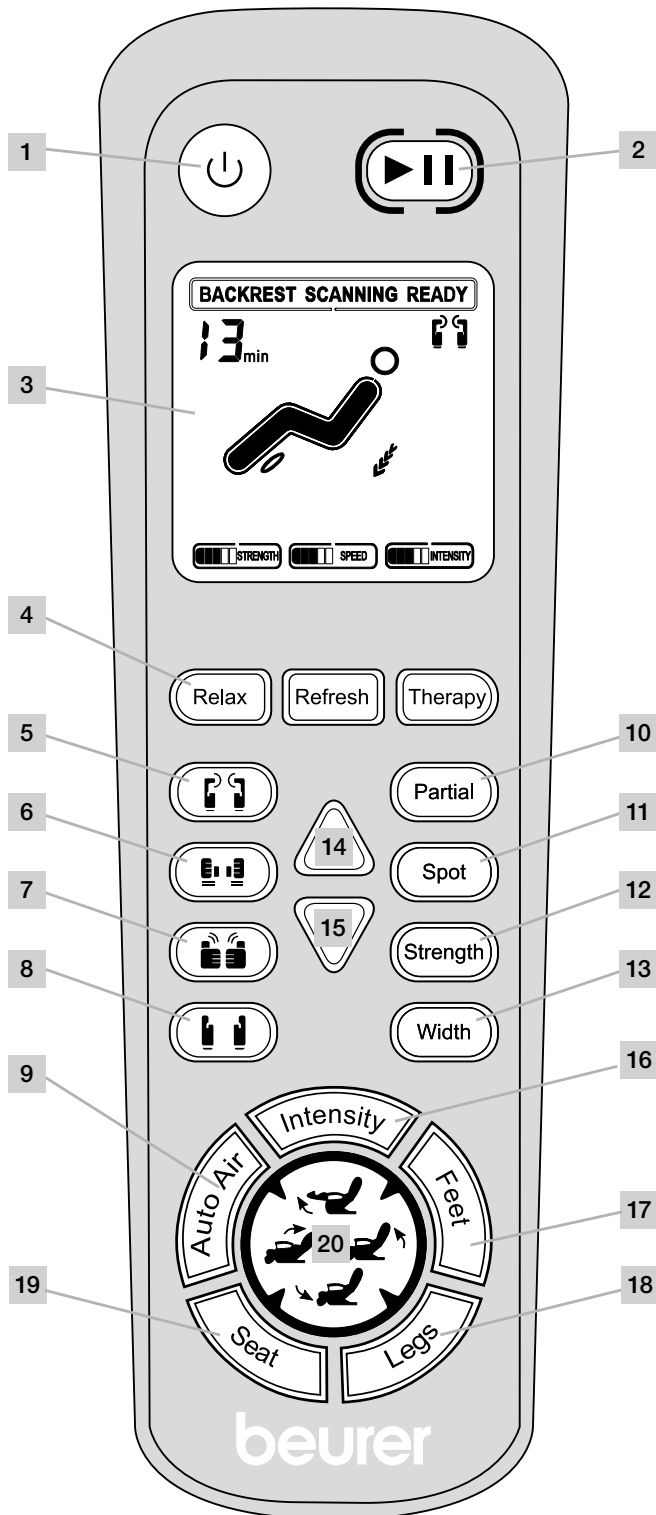- 1 On/off button
- 2 Pause button
- 3 Display
- 4 Automatic massage programmes
- 5 Kneading massage
- 6 Tapping massage
- 7 Shiatsu massage
- 8 Rolling massage
- 9 Air pressure massage auto programme
- 10 Partial massage
- 11 Spot massage
- 12 Strength (strength of the kneading and tapping massage)
- 13 Width (distance between massage rollers)
- 14 Raise roller position
- 15 Lower roller position
- 16 Intensity (air pressure massage)
- 17 Feet (air pressure massage)
- 18 Legs (air pressure massage)
- 19 Seat (air pressure massage)
- 20 Position buttons
  - Raise leg rest
  - Raise back rest
  - Lower leg rest
  - Lower back rest

## Hand switch button functions

### On/off button

- > Press the on/off button .
- The massage chair is ready for operation.*

### Pause button

- > Press the pause button .
- The currently running programme is paused. The massage chair is in standby mode.*
- > Press the pause button  again.
- The programme continues from the point it was paused.*

### Display

The display shows functions that are currently in operation and the current settings.

- The current massage programme
- The current strength and intensity settings
- The current position of the massage rollers
- The remaining duration of the massage

## Massage programmes

- The air pressure massage can be activated or deactivated separately.
- Once the massage programme has finished, the massage chair switches to standby mode.
- The massage can be stopped at any time by pressing the massage button again.
- The massage can be paused at any time by pressing the pause button.

*The currently selected setting is shown on the display.*

## Automatic massage programmes

There are three automatic massage programmes Relax, Refresh and Therapy as well as the air pressure massage.


These automatic programmes massage your entire back with a combination of kneading, tapping, rolling and Shiatsu massage elements.

### Relax


- > Press the relax button .
- The automatic massage programme starts.*
- Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Refresh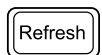

- > Press the refresh button .
- The automatic massage programme starts.*
- Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Therapy

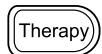- > Press the therapy button .
- The automatic massage programme starts.*
- Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

## Massage programmes

### Kneading

- > Press the massage button .
- The massage programme starts.*
- Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Tapping

- > Press the massage button .
- The massage programme starts.*
- Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Kneading and tapping

> Press the kneading  and tapping  message buttons.

*The kneading and tapping massages can be activated together.*

### Shiatsu

> Press the message button .

*The massage programme starts.*

*Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Roller

> Press the message button .

*The massage programme starts.*

*Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

### Automatic air pressure massage for the seat, calf and foot area.

> Press the auto air button .

*The air pressure massage programme starts.*

*Total duration: 15 minutes. Afterwards, the massage chair switches to standby mode.*

## Concentrated massages

---

### Partial message

While the current massage programme is running:

> Press the partial button .

*The partial message begins with the current position of the massage rollers as the centre point.*

*An area of 8 cm around this centre point is massaged intensively.*

> Press the partial button  again.

*The massage programme continues.*

### Spot message

While the current massage programme is running:

> Press the spot button .

*The spot message begins with the current position of the massage rollers as the centre point.*

*The area directly around this centre point is massaged intensively.*

> Press the spot button  again.

*The massage programme continues.*

## Settings

---

### Strength – strength of the kneading and tapping message

The settings low, medium, high and off are available.

> Press the strength button  in the kneading and tapping automatic message programmes.

*Each time the button is pressed, the next setting is selected.*

### Width – distance between massage rollers

The settings narrow, medium and wide are available.

> Press the width button  in the tapping and rolling message.

*Each time the button is pressed, the next setting is selected.*

### Raise roller position

> Press the up button  in the partial and spot message.

*The massage rollers are raised.*

### Lower roller position

> Press the down button  in the partial and spot message.

*The massage rollers are lowered.*

### Air pressure massage; intensity

The settings low, medium, high and off are available.

> Press the intensity button .

*Each time the button is pressed, the next setting is selected.*

### Air pressure massage, feet

The settings on and off are available.

> Press the feet button .

*Each time the button is pressed, the function is switched on or off.*

### Air pressure massage, legs

The settings on and off are available.

> Press the legs button .

*Each time the button is pressed, the function is switched on or off.*

### Air pressure massage, seat

The settings on and off are available.

> Press the seat button .

*Each time the button is pressed, the function is switched on or off.*

## Position buttons for upright and flat position

---

The functions are available when a massage programme is on or off.

### Raise leg rest position button

> Press and hold the position button .

*The leg rest is raised.*

> Release the position button .

*The leg rest stays in the current position.*

### Raise back rest position button

> Press and hold the position button .

*The back rest is raised.*

> Release the position button .

*The back rest stays in the current position.*

### Lower leg rest position button

> Press and hold the position button .

*The leg rest is lowered.*

> Release the position button .

*The leg rest stays in the current position.*

### Lower back rest position button

> Press and hold the position button .

*The back rest is lowered.*

> Release the position button .

*The back rest stays in the current position.*

## 5. Initial use

### **Warning!**

- Keep packaging material away from children! There is a choking hazard.
- Before using the device for the first time, read the safety notes in the Safety section.
  - ▷ Safety, pages 15 to 16

### **Caution!**

- The massage chair could damage your flooring. Place a suitable, non-slip underlay under the massage chair.

### **Installation location**

- Install the massage chair on a sturdy and even surface.
- Before using the device for the first time, remove all obstacles in front of and behind the massage chair so that the back rest can be lowered and the leg rest raised freely.


### **Unpacking and assembly**

The massage chair MC5000 is delivered in a cardboard box.

### **Caution!**


Do not use sharp or pointed tools (Stanley knife) to remove the packaging.

#### **Step 1:**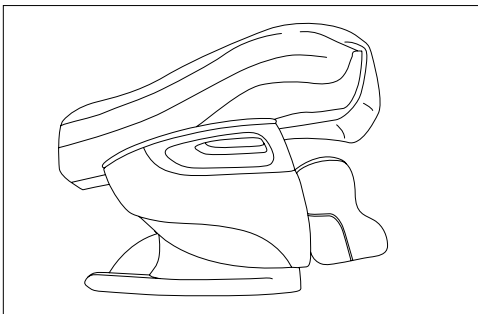


- > Open the cardboard box and hold the massage chair at the designated holding points.
- > Remove the chair from the cardboard box.  
*Four persons are required for this.*
- > Carefully remove all the packaging material

#### **Step 2:**

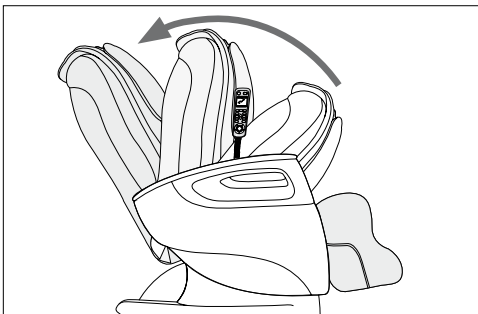
- > Unfold the back rest to the upright position until the locking mechanism audibly engages.  
*The back rest now stands at approx. 115°.*

**Step 3:**

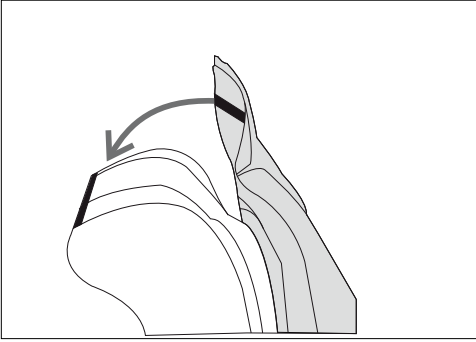- > Connect the lower end of the back cushion and the lower end of the seat cushion using the zip fastening.
- > Lay the seat cushion with the back cushion on the massage chair over the back rest.
- > Connect the back cushion and the back rest using the zip fastening.

**Step 4:**

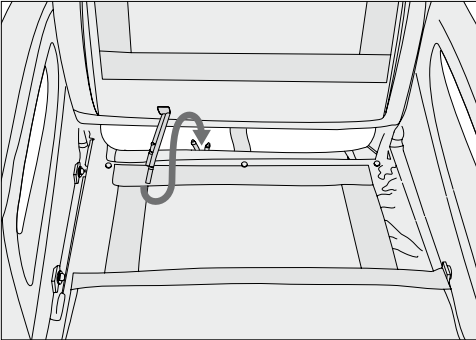- > Fold the seat cushion backwards against the back rest.
- > Connect the air hoses behind the seat with the air hoses on the seat cushion.  
Plug the hose marked red into the connection marked red.  
Plug the hose marked white into the connection marked white.
- > Replace the seat cushion on the seat.

**i** Important! Do not pinch the hoses. Ensure that the hoses are not crossed and or kinked.

**Step 5:**

- > Plug the mains plug into a socket.
- > Switch on the main switch at the bottom of the rear side.
- > Switch on the massage chair at the on/off button (⏻) on the hand switch.

*The massage chair is now ready for operation.*

## 6. Operation

### **Warning!**

- Before using the device for the first time, read the safety notes in the Safety section.
  - ▷ Safety, pages 15 to 16

### **Danger of tipping!**

- Sit down carefully on the chair and only when it is in the upright position. Never sit down on the chair when it is in the flat position.

### **Caution!**

- If you experience irritations of any kind, stop treatment immediately.

### **Important!**

- Never remove the pillow or back cushion during the massage.

## Before switching on

- Wear comfortable clothes during the massage.
- Remove all objects from your trouser pockets (e.g. keys, pocket knife, mobile phone).
- Assume a comfortable, relaxed position.
- Always sit in the middle of the seat so that your spine is positioned precisely between the massage rollers.
- The massage should feel pleasant and relaxing at all times. If the massage is painful or uncomfortable, stop the massage or change your sitting position.
- If the device has not been used for a longer period of time, check that the device is functioning correctly before use.

## Starting the massage


> Press the on/off button  on the hand switch.

*After it is switched on, the massage chair goes into a semi-flat position. You can now select a massage programme.*

- After selecting one of the automatic massage programmes (relax, refresh or therapy), your entire back is scanned to determine the size of the user.

- After selection one of the massage programmes (kneading, tapping, rolling or Shiatsu), the air pressure massage can be activated separately.
- Press the intensity button  to set the strength of the air pressure massage or to switch it off.
- The air pressure massage can be switched on or off separately for the seat, legs and feet.
  - ▷ Massage programme buttons

## Flat position/upright position

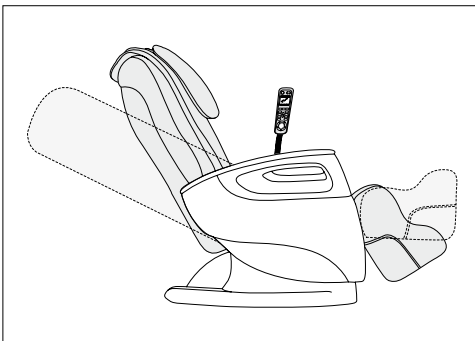- > Put the massage chair in the flat position  
It is possible to set a horizontal flat position.  
The back rest and the leg rest can be raised or lowered individually.  
The functions are available when a massage programme is on or off.

- ▷ Position buttons for upright and flat position

## Switching off the massage chair

After the massage programme is finished, the chair automatically enters standby mode.

### **Caution!**

Do not use the device for longer than 15 minutes at a time otherwise the device could overheat. Allow the device to cool for at least 30 minutes before using it again.


During the massage, you can switch off the device at any time using the on/off button .

*The massage chair returns to the start position. The operating display flashes.*

- > Switch off the massage chair using the main switch beneath the back rest.
- > Pull the mains plug out of the socket.

## Additional settings, operating options

### Folding mechanism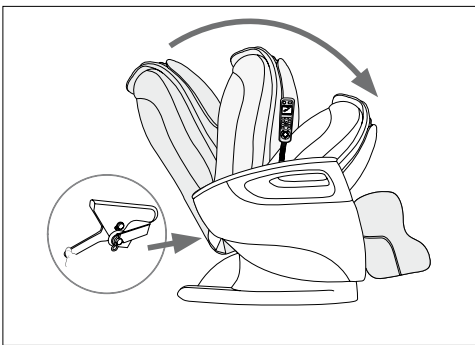


The massage chair can be folded away to save space when stored.

- > Switch off the hand switch at the on/off button .

*The massage chair returns to the start position.*

- > Once the chair has reached the start position, switch off the massage chair using the main switch.
- > Pull the mains plug from the socket.
- > Release the locking mechanism at the back of the back rest at the bottom.
- > Carefully push the back rest forwards on to the seat  
If required, lay the back rest cushion back over the back rest before doing so.

### Transport castors

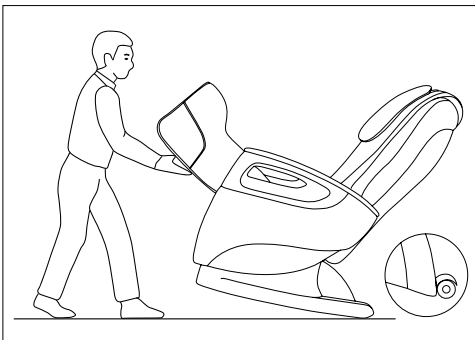The massage chair features castors on the back.

This allows the chair to be transported more easily on level surfaces. Attention! Use an underlay to prevent damage to your flooring.

## 7. Maintenance and storage

### **Warning!**

- The device must be switched off at the main switch after every use and before cleaning.
- Pull the mains plug out of the socket.

- > Wipe the massage chair from time to time with a dry, lint-free or slightly damp cloth.
- > Every two to three months, wipe the chair using a clean cloth with a small amount of care product for artificial leather.
- > Afterwards, dry the chair with a clean, dry cloth.

*Artificial leather is sensitive to grease. Clean the massage chair only using care products designated for artificial leather.*

### **Important!**

- Clean the device using only the methods specified.
- Under no circumstances may liquid enter the device.
- Do not use any solvent-based cleaning products.
- Do not use aggressive cleaning products.  
These can damage the surface.

▷ Folding mechanism, see above

## 8. What if there are problems?

Problem	Cause	Solution
Grinding noises during operation.	These noises are normal and originate from the operating motor or from the massage rollers rubbing against the cushion.	No action is necessary.
The massage cannot be started.	The mains plug is not connected to the socket.	> Plug the mains plug into the socket.
	The massage chair is not switched on at the main switch.	> Switch on the massage chair at the main switch.
	The cutout switch for the massage chair has interrupted the power supply.	> In this case, please contact Customer Service.
Back rest or leg rest cannot be raised or lowered.	Back rest and/or leg rest are overloaded or blocked.	> Remove the objects blocking the device.
The massage function stops when the chair is in operation.	The overheating protection was triggered.	> Switch off the massage chair and allow to cool down for 30 minutes before switching on again.
	The maximum operation period was reached.	

## 9. Technical Data

Product description	Massage chair MC5000
Mains voltage/frequency	220-240 V ~ 50/60 Hz
Energy consumption	200 W
Dimensions (in flat position)	Length 191 cm, width 73 cm, height 64 cm
Weight	Net 61 kg, with packaging 76 kg
Maximum load capacity	200 kg

## 10. Disposal

Observe the local regulations for material disposal. Dispose of the unit in accordance with EC Directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## 11. Warranty and service

In the event of a warranty claim or service case, please contact Customer Services in your respective country.

You can find the address in the enclosed address list.

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