beurer

Phototherapy unit

TL 90

DE	Tageslichtlampe	
	Gebrauchsanweisung.....	2
EN	Daylight therapy lamp	
	Instruction for use.....	10
FR	Lampe de luminothérapie	
	Mode d'emploi	18
SV	Dagsljuslampa	
	Bruksanvisning	26

FI	Kirkasvalolamppu	
	Käyttöohje	33
DA	Dagslys-lampe	
	Bruksanvisning	40
NO	Dagslys-lampe	
	Bruksanvisning	47
RU	Прибор дневного света	
	Инструкция по применению	54

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards
Your Beurer team

Why use a daylight therapy lamp?

When the hours of daylight are noticeably decreased in autumn and people increasingly stay inside, the effects of a lack of light may cumulate. This is often described as „winter depression“. The symptoms can present themselves in a number of ways:

- Imbalance
- Subdued mood
- Lack of energy and listlessness
- Generally feeling under the weather
- Need for more sleep
- Loss of appetite
- Difficulty concentrating

The cause of these symptoms is the fact that light, particularly sunlight, is essential for life and has a direct effect upon the human body. Sunlight indirectly controls the production of melatonin, which is released into the blood only in darkness. This hormone tells the body when it is time to sleep. In the months when there is less sunlight, therefore, production of melatonin increases. This makes it harder to get up in the morning, since the body’s functions are shut down. If you use the daylight therapy lamp immediately after waking up in the morning—thus as early as possible—you can end the production of melatonin so that your mood takes a positive turn. In addition, light deficiency also hinders production of the happiness hormone serotonin, which decisively affects our well-being. So application of light involves quantitative changes in hormones and neurotransmitters in the brain, which can affect our level of activity, our feelings and our well-being. Light can help reverse this hormone imbalance. To compensate for such a hormonal imbalance, the daylight therapy lamp can create a suitable replacement for natural sunlight.

In the medical field, daylight therapy lamps are used to compensate for the effects of a lack of light. Daylight therapy lamps simulate daylight over 10,000 lux. The light can influence the human body and be used as a treatment or as a preventative measure. Normal electric light is not sufficient to influence the hormonal balance. Because in a well-lit office the illuminance level amounts to just 500 lux.

Content

1. Included in delivery	11	7. Usage	13
2. Signs and symbols	11	8. Cleaning and maintenance	15
3. Intended use	11	9. What if there are problems?	15
4. Warnings and safety notes	11	10. Disposal	15
5. Device description	12	11. Technical specifications	15
6. Initial use	13	12. Warranty / Service	16

1. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x Daylight therapy lamp
- 1 x These operating instructions

2. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Warning Warning instruction indicating a risk of injury or damage to health		The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products
	Important Safety note indicating possible damage to the device/accessory		Permissible storage temperature and humidity
	Product information Note on important information		Permissible operating temperature and humidity
	Observe the instructions Read the instructions before starting work and/or operating devices or machines	IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
	Manufacturer	SN	Serial number
	Protection class II device The device is double-protected and thus corresponds to protection class 2		Separate the packaging elements and dispose of them in accordance with local regulations.
	OFF / ON		Marking to identify the packaging material. A = Material code, B = Material number: 1-7 = Plastics, 20-22 = Paper and cardboard

3. Intended use

The daylight therapy lamp is intended for radiation on human bodies only. Do not use the daylight therapy lamp on animals. The device is only intended for domestic/private use. The device is not intended for commercial use.

4. Warnings and safety notes

Safety notes

Warning

- The light of the daylight therapy lamp is only intended for use on the human body.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Also ensure that the daylight therapy lamp has a firm footing.
- The mandatory minimum distance between the daylight therapy lamp and flammable objects is 1.0 m.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness); do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).

- Do not cover up or pack away the unit while it is warm.
- Always remove the mains plug and let the unit cool off before you touch it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Keep the mains cable away from hot objects and naked flames.
- Protect the unit from strong impacts.
- Do not pull the mains plug from the socket by the cord.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- Do not use the device in the presence of flammable anaesthetic gas connections with air, oxygen or nitrogen oxide.
- No calibration and no preventative checks or maintenance is to be carried out on this device.
- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- As the patient/user, you can operate all functions of this device.

General notes

⚠ Important

- If you are taking medication such as pain relievers, antihypertensives or antidepressants, consult your doctor before using the daylight therapy lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before beginning to use the daylight therapy lamp.
- Please do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humour.
- Always consult a doctor before using the daylight therapy lamp if you have a strong sensitivity to light or you are prone to migraine attacks.
- If you have health concerns of any kind, contact your general practitioner!
- Before you use the unit for the first time, remove all packaging materials.
- The lamp tubes are excluded from the warranty.

Repairs

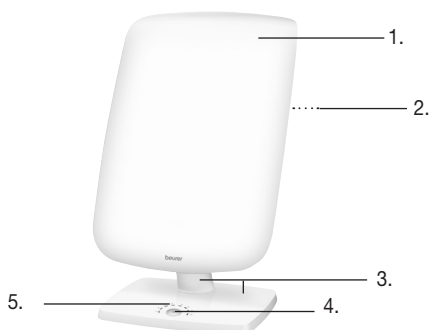
⚠ Important

- Do not open the device. Please do not attempt to repair the unit yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.

5. Device description

Overview

1. Fluorescent screen
2. Rear of the device housing
3. Stand
4. On/Off button
5. LED display

6. Initial use

Take the device out of the plastic wrapping. Check the device for damage or faults. If you notice any damage or faults on the device, do not use it and contact customer services or your supplier.

Positioning the device

Place the device on an even surface. The position should be chosen to ensure a distance of between 15 cm and 50 cm between the user and the device. The lamp is most effective at this distance.

Mains connection

Connect the unit only to the mains voltage listed on the type plate. Push the plug completely into the mains socket.

Note:

- Ensure that there is a socket near the place where the lamp will stand.
- Lay the mains cable in such a way that no one can trip over it.

7. Usage

1	Switching on the lamp ▶ Press the On/Off button. The tubes turn on. Set the angle of the fluorescent screen in accordance with your individual requirements. ATTENTION! The brightness of the tubes may be visibly different when switched on. The tubes develop their full luminosity after a couple of minutes.
2	Enjoying the light Sit as close as possible to the lamp, between 15 cm and 50 cm. You can go about your normal activities while using it. You can read, write, make telephone calls, etc. <ul style="list-style-type: none">• Every so often, briefly look directly into the light, since the effect results from the eyes/ retina receiving the light.• Use the daylight therapy lamp as often as you want. However, the treatment is at its most effective if you carry out the light therapy according to the prescribed time period of at least 7 successive days.• The most effective time of day for the treatment is between 6am and 8pm and we recommend that it is used for 2 hours per day.• Do not look directly into the light the whole time you are using it, however, since that could lead to overstimulation of the retina.• Start with shorter periods of illumination, which you increase over the course of a week. Note: Eye aches and headaches may arise after the first times using the lamp. These will go away in later sessions, as the nervous system will become accustomed to the new stimulation.

<p>3</p>	<p>LED display</p> <p>The daylight therapy lamp TL90 indicates your current treatment time with the aid of 8 LEDs. When the device is switched on, all LEDs will briefly light up. After a treatment time of 15 minutes, one LED will light up at a time.</p> <table border="1" data-bbox="140 199 789 518"> <thead> <tr> <th>Treatment time</th> <th>Number of lit LEDs</th> </tr> </thead> <tbody> <tr> <td>0 – 15 minutes</td> <td>0</td> </tr> <tr> <td>16 – 30 minutes</td> <td>1</td> </tr> <tr> <td>31 – 45 minutes</td> <td>2</td> </tr> <tr> <td>46 – 60 minutes</td> <td>3</td> </tr> <tr> <td>61 – 75 minutes</td> <td>4</td> </tr> <tr> <td>76 – 90 minutes</td> <td>5</td> </tr> <tr> <td>91 – 105 minutes</td> <td>6</td> </tr> <tr> <td>106 – 120 minutes</td> <td>7</td> </tr> <tr> <td>>120 minutes</td> <td>8</td> </tr> </tbody> </table> <p>The maximum recommended treatment time will pass after 120 minutes and all LEDs will begin to flash 5 times. After the 120 minutes has passed, switch off the daylight therapy lamp by pressing the ON/OFF button.</p>	Treatment time	Number of lit LEDs	0 – 15 minutes	0	16 – 30 minutes	1	31 – 45 minutes	2	46 – 60 minutes	3	61 – 75 minutes	4	76 – 90 minutes	5	91 – 105 minutes	6	106 – 120 minutes	7	>120 minutes	8
Treatment time	Number of lit LEDs																				
0 – 15 minutes	0																				
16 – 30 minutes	1																				
31 – 45 minutes	2																				
46 – 60 minutes	3																				
61 – 75 minutes	4																				
76 – 90 minutes	5																				
91 – 105 minutes	6																				
106 – 120 minutes	7																				
>120 minutes	8																				
<p>4</p>	<p>Important instructions</p> <p>When using the lamp, maintain the recommended distance of 15 – 50 cm between your face and the lamp. The duration of the application depends on the distance:</p> <table border="1" data-bbox="140 721 868 869"> <thead> <tr> <th>Lux</th> <th>Distance</th> <th>Application duration</th> </tr> </thead> <tbody> <tr> <td>10.000</td> <td>approx. 15 cm</td> <td>0,5 hour</td> </tr> <tr> <td>5.000</td> <td>approx. 30 cm</td> <td>1 hour</td> </tr> <tr> <td>2.500</td> <td>approx. 50 cm</td> <td>2 hours</td> </tr> </tbody> </table> <p>Basically, the closer you are to the source of light, the less time you should use it.</p>	Lux	Distance	Application duration	10.000	approx. 15 cm	0,5 hour	5.000	approx. 30 cm	1 hour	2.500	approx. 50 cm	2 hours								
Lux	Distance	Application duration																			
10.000	approx. 15 cm	0,5 hour																			
5.000	approx. 30 cm	1 hour																			
2.500	approx. 50 cm	2 hours																			
<p>5</p>	<p>Enjoying light over longer periods</p> <p>Repeat your use of the lamp in the dark seasons for at least 7 successive days, or even longer, depending on your individual needs. If possible, conduct the treatment in the morning hours.</p>																				
<p>6</p>	<p>Switching off the lamp</p> <p>▶ Press the On/Off button. The tubes turn off. Pull the mains plug from the socket.</p> <p>CAUTION!</p> <p>The lamp is hot after being used. Let the lamp cool off first long enough before you put it away and/or pack it up!</p>																				

Replacing the lamp tubes/starter

The service life of the fluorescent tubes is on average 10,000 hours. If one tube needs replacing, both fluorescent tubes must be replaced with new tubes of the same type. For this purpose, contact a retailer, an authorised service centre or appropriately qualified personnel or contact the customer service.

The following tubes are approved as replacement:

- PLL 36W 2G11
- The manufacturer's tubes are available with the order number 162.942 from customer support

Warning

- For environmental reasons, do not dispose of the fluorescent tubes with the household waste. Dispose of the unit at a suitable local collection or recycling point. Please contact the local authorities responsible for waste disposal.
- Unplug the device, to avoid the risk of electrocution.
- Take care when handling fluorescent tubes to avoid the risk of cut injuries.

8. Cleaning and maintenance

The unit should be cleaned occasionally.

Important

- Ensure that no water penetrates inside the unit!
Before you clean the unit, always switch it off, unplug it and let it cool off.
- Do not wash the unit in a washing machine!
Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
Do not use abrasive detergents and never immerse the unit in water.

Storage

If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

Follow the storage instructions provided in the “Technical specifications” chapter.

9. What if there are problems?

Problem	Possible cause	Solution
Device does not light up	On/off button switched off	Switch on the on/off button.
	No electricity	Connect the mains plug correctly.
	No electricity	The mains cable is defective. Contact customer services or your retailer.
	Fluorescent tube has reached the end of its service life. Fluorescent tube is defective.	If you need to have the unit repaired, contact customer service or an authorised dealer.

10. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

11. Technical specifications

Dimensions (L/W/H)	33,7 x 24,1 x 58,1 cm
Weight	~ 3 kg
Lighting elements	PLL 36W 2G11
Power	2 x 36 Watts
Light intensity	10.000 lux (Application approx. 15 cm)
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.
Mains connection	230 V ~, 50 Hz
Operating conditions	+5°C to +45°C, 30 – 90 % relative humidity; 700 – 1060 hPa ambient pressure

Storage conditions	+5°C to +50°C, 30 – 90 % relative humidity; 700 – 1060 hPa ambient pressure
Product classification	Protection class II
Degree of efficiency of the ballast	~ 98%
Source efficiency (lm/W) of the bulb	64 lm/W
Color temperature of tubes	6.500 Kelvin
Fuse type	F2AL, 250V

Subject to technical modifications.

The serial number is located on the device or in the battery compartment.

Brightness: 10,000 Lux (this light intensity value is just for information and regarding IEC 60601-2-57, this light source is classified as an exempt group).

Notes on electromagnetic compatibility

The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz) and the European standard EN 60601-1-2 (in accordance with CISPR 11, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to particular precautions with regard to electromagnetic compatibility.

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

12. Warranty / Service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as "Beurer") provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached "International Service" list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Taishan Anson Electrical Appliances Co. Ltd • Dragon Mountain Industrial Estate
BEURER GmbH • Söflinger Str. 218 • 89077 Ulm, Germany
Share Info GmbH • Heerdter Lohweg 83 • 40549 Düsseldorf, Germany