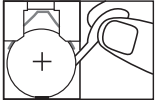
- D** **Personenwaage**  
Gebrauchsanleitung
- GB** **Bathroom scale**  
Instruction for Use
- F** **Pèse-personne**  
Mode d'emploi
- E** **Báscula de baño**  
Instrucciones para el uso
- I** **Bilancia pesapersona**  
Istruzioni per l'uso
- TR** **Terazi**  
Kullanma Talimatı
- RUS** **Напольные весы**  
Инструкция по применению
- PL** **Waga osobowa**  
Instrukcja obsługi
- NL** **Personenweegschaal**  
Gebruikershandleiding
- P** **Balança de banho**  
Instruções de utilização
- GR** **Οδηγίες χρήσης**  
Οδηγίες χρήσης
- DK** **Personvægt**  
Betjeningsvejledning
- S** **Personvåg**  
Bruksanvisning
- N** **Personvekt**  
Bruksveiledning
- FIN** **Henkilövaaka**  
Käyttöohje
- CZ** **Osobní váha**  
Návod k použití
- SLO** **Osebna tehtnica**  
Navodila za uporabo
- H** **Személmérleg**  
Használati utasítás
- RO** **Cântar pentru persoane**  
Instrucțiuni de utilizare
- ميزان الأشخاص  
دليل الاستعمال

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- (GB)** Getting started
- (F)** Mise en service
- (E)** Puesta en marcha
- (I)** Messa in funzione
- (NL)** Ingebruikname

- (P)** Colocação em funcionamento
- (TR)** İlk çalıştırma
- (GR)** Έναρξη λειτουργίας
- (DK)** Ibrugtagning

- (S)** Börja använda vågen
- (N)** Bruk
- (FIN)** Käyttöönotto
- (RUS)** Ввод в эксплуатацию
- (PL)** Uruchomienie

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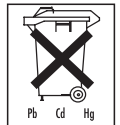
### (GB) General advice

- Max. capacity 150 kg (330 lb / 24 st), Graduation 100 g (0.2 lb / 1/4 st).
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers. You are legally obliged to dispose of the batteries.
- Note: Batteries containing pollutant substances are marked as follows:  
 Pb= Battery contains lead,  
 Cd = Battery contains cadmium,  
 Hg = Battery contains mercury.

- Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Regulation 2002/96/EC-WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.

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