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## EN Important instructions

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

### Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Observe the instructions for use		Dispose of packaging in an environmentally friendly manner
	Manufacturer		The product can be recycled, is subject to extended producer responsibility, and is collected separately.

- Max. capacity 136 kg, Graduation 500 g.
- Place the scales on a firm level surface; a firm surface is essential for correct measurement
- Protect your personal scales from impact with hard objects, moisture, dust, chemicals, toiletries, liquid cosmetics, great temperature fluctuation and closeness to sources of heat (open fires, radiators).
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water. Do not use aggressive or abrasive cleaning products.
- Storage: Do not place any objects on the scale when not in use.
- The scale is not intended to be used for commercial purposes.
- Remove transport safety catch.
- Never step onto the outer edge of the scales with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Dispose of packaging in an environmentally friendly manner.

### Warranty/service

Further information on the warranty and warranty conditions can be found in the warranty leaflet supplied.

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