Glaswaage

Typ: GS 58

GB Glass scale
F Pèse-personne en verre
E Báscula de vidrio para personas
I Bilancia in vetro

NL Weegschaal met glazen weegplateau
P Balança de vidro
TR Cam Terazi
GR Γυάλινη-υγαριά
DK Glasvægt

S Glasvåg
N Glassvekt
FIN Lasivaaka
RUS Стеклянные весы
PL Waga szklana
CZ Skleněná váha

SLO Steklena tehnica
HU Üvegmérleg
RO Cântar de sticla
الميزان الزجاجي

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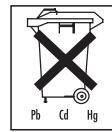
General advice

- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic fields and sources of heat that are too near.
- Never step onto the outer edge of the scale with one foot as this could cause the scales to tip!
- Keep packaging material away from children!
- Not intended for commercial use.
- Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Customer Service.
- Before making a claim, please check the batteries and replace them if necessary.
- Repairs must only be carried out by Customer Services or authorised suppliers.
- Spent batteries and rechargeable batteries do not constitute normal household waste! They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers. You are legally obliged to dispose of the batteries.
- Note: Batteries containing pollutant substances are marked as follows:
Pb = Battery contains lead,
Cd = Battery contains cadmium,
Hg = Battery contains mercury.
- Follow local regulations regarding the disposal of materials. The appliance should be disposed of according to Regulation 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment). In case of queries, contact the municipal authorities responsible for waste disposal in your area.

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