DE	Tageslichtlampe Gebrauchsanweisung.....	2	DA	Dagslys Lampe Brugsanvisning.....	37
EN	Daylight therapy lamp Instruction for use.....	9	NO	Dagslys Lampe Bruksanvisning.....	44
FR	Lampe de luminothérapie Mode d'emploi.....	16	RU	Прибор дневного света Инструкция по применению.....	50
SV	Dagsljuslampe Bruksanvisning.....	23			
FI	Kirkasvalolamppu Käyttöohje.....	30			

Inhalt

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Germany

Contents

1. About the daylight therapy lamp	9	8. Storage	13
2. Signs and symbols	10	9. Disposal.....	13
3. Notes	10	10. What if there are problems?	13
4. Device description.....	11	11. Technical specifications	13
5. Initial use	11	12. Mains part.....	14
6. Operation.....	12	13. Warranty/service.....	15
7. Cleaning and care of the device.....	13		

1. About the daylight therapy lamp

Dear valued customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

With kind regards
Your Beurer team

Why use a daylight therapy lamp?

When the hours of daylight are noticeably decreased in autumn and people increasingly stay inside, the effects of a lack of light may cumulate. This is often described as „winter depression“. The symptoms can present themselves in a number of ways:

- Imbalance
- Subdued mood
- Lack of energy and listlessness
- Generally feeling under the weather
- Need for more sleep
- Loss of appetite
- Difficulty concentrating

The cause of these symptoms is the fact that light, particularly sunlight, is essential for life and has a direct effect upon the human body. Sunlight indirectly controls the production of melatonin, which is released into the blood only in darkness. This hormone tells the body when it is time to sleep. In the months when there is less sunlight, therefore, production of melatonin increases. This makes it harder to get up in the morning, since the body’s functions are shut down. If you use the daylight therapy lamp immediately after waking up in the morning—thus as early as possible—you can end the production of melatonin so that your mood can take a positive turn. In addition, light deficiency also hinders production of the happiness hormone serotonin, which decisively affects our well-being. So application of light involves quantitative changes in hormones and neurotransmitters in the brain, which can affect our level of activity, our feelings and our well-being. Light can help reverse this hormone imbalance. To compensate for such a hormonal imbalance, the daylight therapy lamp can create a suitable replacement for natural sunlight.


In the medical field, daylight therapy lamps are used to compensate for the effects of a lack of light. Daylight therapy lamps simulate daylight over 10,000 lux. The light can influence the human body and be used as a treatment or as a preventative measure. Normal electric light is not sufficient to influence the hormonal balance. Because in a well-lit office the illuminance level amounts to just 500 lux.

Included in delivery:

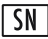- Daylight therapy lamp
- Holder
- Storage pouch
- Mains part
- These operating instructions

2. Signs and symbols

The following symbols appear in these instructions or on the device.

	Read the instructions of use	 Note	Note on important information
 Warning	Warning instruction indicating a risk of injury or damage to health		Manufacturer
 Important	Safety note indicating possible damage to the device/accessory		Device in protection class 2
	Serial number		OFF / ON
 0123	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products	IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water
Storage 	Permissible storage temperature and humidity	 PAP 	Dispose of packaging in an environmentally friendly manner
Operating 	Permissible operating temperature and humidity		

3. Notes

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Safety notes

Warning

- The light of the daylight therapy lamp is only intended for use on the human body.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the device or accessories. When in doubt, do not use the device and contact your dealer or the customer service address provided.
- Also ensure that the daylight therapy lamp has a firm footing.
- Connect the device only to the mains voltage listed on the type plate.
- Do not dip the device into water and do not use it in wet rooms.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness); do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover up or pack away the device while it is warm.
- Always unplug the mains part and allow the device to cool down before touching it.
- Do not touch the device with wet hands while it is plugged in; do not allow any water to be sprayed onto the device. The device must be operated only when it is completely dry.
- Ensure that you only insert and remove the mains part with dry hands and that you only press the ON/OFF button with dry hands.
- Keep the mains cable away from hot objects and naked flames.
- Protect the device from strong impacts.
- Do not pull the mains part out of the socket using the mains cable.
- Do not use the device if it shows signs of damage or does not function properly. In these cases, contact Customer Services.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.

- Disconnection from the power supply network is only guaranteed when the mains part is unplugged.
- Do not use the device in the presence of flammable anaesthetic gas connections with air, oxygen or nitrogen oxide.
- No calibration and no preventative checks or maintenance is to be carried out on this device.
- You cannot repair the device. The device contains no parts that you can repair.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- To avoid strangulation and entanglement, keep cable out of reach of young children.
- The patient is the intended operator. The patient can use the device and accessories according to this manual.

General notes

⚠ Important

- If you are taking medication such as pain relievers, antihypertensives or antidepressants, consult your doctor before using the daylight therapy lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before beginning to use the daylight therapy lamp.
- Please do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humour.
- Always consult a doctor before using the daylight therapy lamp if you have a strong sensitivity to light or you are prone to migraine attacks.
- If you have health concerns of any kind, contact your general practitioner!
- Before you use the device for the first time, remove all packaging materials.
- Light sources are excluded from the guarantee.
- After storage or transport condition, before use keep the device in ambient room for at least 2 hours.

Repairs




⚠ Important

- Do not open the device. Please do not attempt to repair the device yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If you need to have the device repaired, contact customer service or an authorised dealer.

4. Device description

Overview

1. Fluorescent screen
2. Rear of the device housing
3. Holder
4. ON/OFF button
5. Mains part connection
6. Storage slot for holder

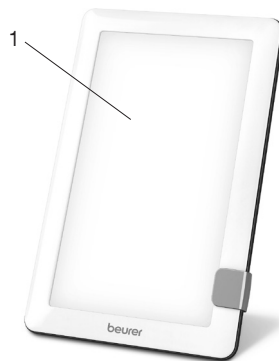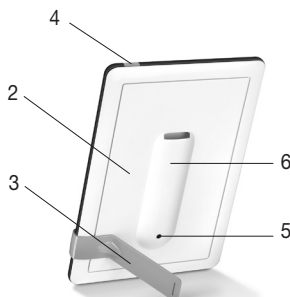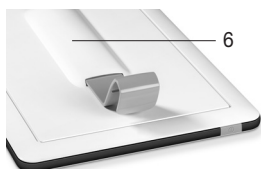5. Initial use

Take the device out of the plastic wrapping. Check the device for damage or faults. If you notice any damage or faults on the device, do not use it and contact customer services or your supplier.

Positioning the device

Place the device on an even surface. The position should be chosen to ensure a distance of between 10 cm and 20 cm between the user and the device. The lamp is most effective at this distance.

Mains connection

- To prevent possible damage to the device, the daylight therapy lamp must only be used with the mains part described here.
- Insert the mains part into the connection provided for this purpose on the rear of the daylight therapy lamp. The mains part must only be connected to the mains voltage that is specified on the type plate.
- After using the daylight therapy lamp, unplug the mains part from the mains socket first and then disconnect it from the daylight therapy lamp.

Note:

- Ensure that there is a socket near the place where the lamp will stand.
- Lay the mains cable in such a way that no one can trip over it.

6. Operation

1	Remove the holder from the storage slot on the rear of the device.												
2	Clip on the holder at the side. The orientation/angle of the TL 30 changes depending on where you attach the holder to the device. This allows you to tailor the angle as desired. The holder can be attached on both the long and short sides, which enables you to use the TL 30 both horizontally and vertically.												
3	Switching on the lamp ▶ Press the On/Off button.												
4	Enjoying the light Sit as close as possible to the lamp, between 10 cm and 20 cm. You can go about your normal activities while using it. You can read, write, make telephone calls, etc. <ul style="list-style-type: none">• Every so often, briefly look directly into the light, since the effect results from the eyes/ retina receiving the light.• Use the daylight therapy lamp as often as you want. However, the treatment is at its most effective if you carry out the light therapy according to the prescribed time period of at least 7 successive days.• The most effective time of day for the treatment is between 6am and 8pm and we recommend that it is used for 2 hours per day.• Do not look directly into the light the whole time you are using it, however, since that could lead to overstimulation of the retina.• Start with shorter periods of illumination, which you increase over the course of a week. Note: Eye aches and headaches may arise after the first times using the lamp. These will go away in later sessions, as the nervous system will become accustomed to the new stimulation.												
5	Important instructions When using the lamp, maintain the recommended distance of 10 - 20 cm between your face and the lamp. The duration of the application depends on the distance: <table border="1" data-bbox="140 1098 770 1241"><thead><tr><th>Lux</th><th>Distance</th><th>Application duration</th></tr></thead><tbody><tr><td>10.000</td><td>approx. 10 cm</td><td>0,5 hour</td></tr><tr><td>5.000</td><td>approx. 15 cm</td><td>1 hour</td></tr><tr><td>2.500</td><td>approx. 20 cm</td><td>2 hours</td></tr></tbody></table> <p>Basically, the closer you are to the source of light, the less time you should use it.</p>	Lux	Distance	Application duration	10.000	approx. 10 cm	0,5 hour	5.000	approx. 15 cm	1 hour	2.500	approx. 20 cm	2 hours
Lux	Distance	Application duration											
10.000	approx. 10 cm	0,5 hour											
5.000	approx. 15 cm	1 hour											
2.500	approx. 20 cm	2 hours											
6	Enjoying light over longer periods Repeat your use of the lamp in the dark seasons for at least 7 successive days, or even longer, depending on your individual needs. If possible, conduct the treatment in the morning hours.												

7	<p>Switching off the lamp</p> <p>▶ Press the On/Off button. The LEDs switch off. Unplug the mains part from the mains socket.</p> <p>CAUTION!</p> <p>The light remains hot after use. Let the lamp cool off first long enough before you put it away and/or pack it up!</p>
----------	---

7. Cleaning and care of the device

The device should be cleaned occasionally.

Important

- Ensure that no water penetrates inside the device!
Before you clean the device, always switch it off, unplug it and let it cool off.
- Do not wash the device in a dishwasher!
Use a slightly damp cloth to clean the device.
- Do not touch the device with wet hands while it is plugged in; do not allow any water to be sprayed onto the device. The device must be operated only when it is completely dry.
Do not use abrasive detergents and never immerse the device in water.

8. Storage

If you are not going to use the device for an extended period of time, disconnect it and store it in a dry place, out of the reach of children.

Follow the storage instructions provided in the “Technical data” chapter.

9. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

10. What if there are problems?

Problem	Possible cause	Solution
Device does not light up	On/off button switched off	Switch on the on/off button.
	No electricity	Connect the mains part properly.
	No electricity	The mains part is faulty. Contact customer services or your retailer.
	LEDs have reached the end of their service life. LEDs faulty.	If you need to have the device repaired, contact customer service or an authorised dealer.

11. Technical specifications

Model no.	GCE501
Type	TL 30 G
Dimensions (WHD)	156 x 236 x 20 mm
Weight	approx. 320 g (approx. 420 g incl. mains adapter + storage pouch)
Lighting elements	LED
Power	7,8 watts
Light intensity	10,000 lux (Distance: approx. 10 cm)
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.

Operating conditions	0°C to +35°C, 15 - 90% relative humidity, 700 - 1060 hPa ambient pressure
Transport and storage conditions	-20°C to +60°C, 15 - 90% relative humidity, 700 - 1060 hPa ambient pressure
Product classification	External power supply, Protection class II, IP21
Color temperature of LEDs	6,500 kelvin
Included in delivery	Daylight therapy lamp, holder, storage pouch, mains part, these operating instructions
Limit short wavelength	440 nm
Expected life of the device	1,000 hours

Maximum output of optical radiation for the TL 30

Radiance Output	Risk group classified by IEC 60601-2-57: 2011	Max. value
E _{UVA} : Eye UV-A	Exempt Group	0,3 W·m ⁻²
ES: Actinic UV skin & eye	Exempt Group	0 W·m ⁻²
EIR: Infrared radiation hazard exposure limits for the eye	Exempt Group	0 W·m ⁻²
LIR: Retinal thermal (mild visual irritation)	Exempt Group	9,5 W·m ⁻² ·sr ⁻¹
LB: Blue light	Exempt Group	21 W·m ⁻² ·sr ⁻¹
LR: Retinal thermal	Exempt Group	1065 W·m ⁻² ·sr ⁻¹

Subject to technical modifications.

The serial number is located on the device or in the battery compartment.

Brightness: 10,000 Lux (this light intensity value is just for information and regarding IEC 60601-2-57, this light source is classified as an exempt group).

Notes on electromagnetic compatibility

The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act and the European standard EN 60601-1-2 (in accordance with IEC61000-3-2, IEC61000-3-3) and is subject to particular precautions with regard to electromagnetic compatibility. The device is suitable for use in all environments listed in these instructions for use, including domestic environments.

- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

12. Mains part

Model no.	LXCP12-012650BEH
Input	100–240 V ~ 50/60 Hz
Output	12 V DC, 650 mA
Protection	The device is double protected and therefore corresponds to protection class 2.
	Polarity of the the DC voltage connection
Classification	IP21, protection class II

13. Warranty/service

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international".

In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

1. The warranty period for BEURER products is either 3 years or- if longer- the country specific warranty period from date of Purchase.
In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages because of
 - a. improper treatment, e.g. nonobservance of the user instructions.
 - b. repairs or tampering by the customer or unauthorised third parties.
 - c. transport from the manufacturer to the consumer or during transport to the service centre.
 - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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Globalcare Medical Technology Co., Ltd.
Distributed by BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)
Donawa Lifescience Consulting Srl, Piazza Albania, 10, 00153 Rome, Italy