beurer

HK 70

- DE** HK 70 Rückenstütze mit Wärme
Gebrauchsanweisung 2
- EN** HK 70 Back rest with heat
Instructions for use 13
- FR** Dossier avec diffusion de chaleur
HK 70
Mode d'emploi 23
- ES** HK 70 Respaldo calefactado
Instrucciones de uso 34
- IT** Sostegno per la schiena con
riscaldamento HK 70
Istruzioni per l'uso 45
- TR** HK 70 Isıtmalı sırt desteği
Kullanım kılavuzu 55
- RU** HK 70 Ортопедическая
подушка под спину
с функцией прогрева
Инструкция по
применению 65
- PL** HK 70 Podpórka pod plecy z
ogrzewaniem
Instrukcja obsługi 75
- DA** HK 70 rygstøtte med varme
Betjeningsvejledning 86
- SV** HK 70 Ryggdyna med värme
Bruksanvisning 96
- NO** HK 70 Ryggstøtte med varme
Bruksanvisning 106
- FI** HK 70 Lämmitettävä selkätuki
Käyttöohje 116

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Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, air and baby.

With kind regards
Your Beurer team

Contents

1. Signs and symbols	13	5.4 Setting the temperature	19
2. Included in delivery	14	5.5 Automatic switch-off	19
2.1 Device description	14	5.6 Switching the device off	19
3. Important information		6. Cleaning and maintenance	19
Retain for future reference	15	7. Storage	20
4. Intended use	16	8. Disposal	21
5. Operation	16	9. What if there are problems?	21
5.1 Safety	16	10. Technical data	21
5.2 Initial use	17	11. Warranty/service	21
5.3 Switching the device on	18		

1. Signs and symbols

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

	Read instructions!		Extra-delicate cycle at 30°C
			Do not bleach
	Do not insert needles!		Do not tumble dry
			Do not iron
	Do not use when folded or bunched up!		Do not dry clean
			Do not wash
	Must not be used by very young children (0-3 years old).		The fabrics used in this device meet the high ecological requirements of Oeko-Tex Standard 100 as verified by the Hohenstein Research Institute.
	Dispose of packaging in an environmentally friendly manner		

	Direct current		Manufacturer
	The unit is double-protected and thus corresponds to protection class 2.		
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE.		
	This product satisfies the requirements of the applicable European and national directives.		
	Certification symbol for products that are exported to the Russian Federation and members of the CIS		
	WARNING; warning instruction indicating a risk of injury or damage to health		
	IMPORTANT; safety note indicating possible damage to the device/accessory		
	NOTE; note on important information		


2. Included in delivery

Check that the exterior of the cardboard delivery packaging is intact and make sure that all contents are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

- 1 x back rest with heat
- 1 x mains part
- 1 x car adapter

2.1 Device description

- 1 Heat pad
- 2 Mains part
- 3 Car adapter
- 4 Hook-and-loop fastening bands for attaching to the backrest of the chair
- 5 Plug-in coupling
- 6 ON/OFF button with LED display
- 7 Car fastening straps

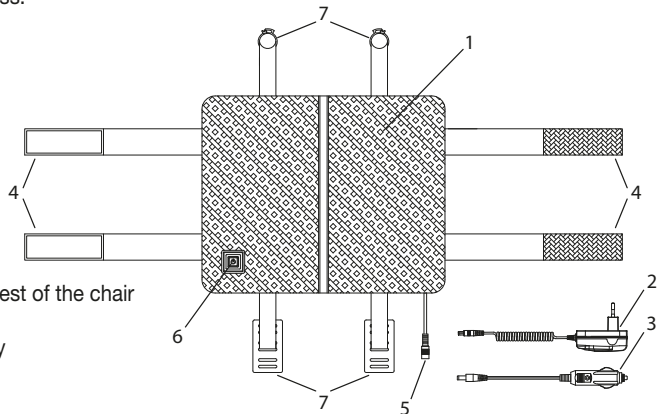3. Important information

Retain for future reference

WARNING

- Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.
- This heating pad must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- This heating pad may not be used for very young children under the age of three (0-3) due to their inability to react to overheating.
- This heating pad may not be used for young children over the age of three (3-8) unless the control has been pre-set by a parent or guardian and unless the child has been adequately instructed how to operate the heating pad safely.
- This heating pad can be used by older children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Children must not play with the heating pad.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This heating pad is not intended for use in hospitals.
- The device is only intended for domestic/private use, not for commercial use.
- Do not insert pins.
- Do not use folded or rucked.
- Do not use if wet.
- This heat pad must only be used with the supplied mains part and car adapter.
- Connect the unit only to the mains voltage listed on the rating label.

- The electrical and magnetic fields emitted by this heating pad may interfere with the function of a pacemaker. However, they are still well within the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Do not pull, twist or make sharp kinks in the cables.
- This heating pad must be examined frequently for signs of wear or damage. If there are such signs, if the heating pad has been misused or does not work, return it to the supplier before switching it on again.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to adhere to this instruction will void the warranty.
- If the spiral cable or the mains adapter of this heat pad are damaged, it must be replaced by the manufacturer, its customer service department or a similarly qualified person to avoid any risk.
- When this heating pad is switched on:
 - Do not place any sharp objects on it.
 - Do not place any heat sources, such as hot water bottles, heat pads or similar, on it.
- The electronic components in the controller warm up when the heating pad is in use. For this reason, the control must never be covered or placed on the heating pad when it is in use.
- It is essential to observe the information relating to the following chapters: Operation, Cleaning and maintenance, Storage.
- Should you still have questions on using our appliances, please contact our customer service.

4. Intended use

IMPORTANT

This heat pad is only designed to warm up the human body.

5. Operation

5.1 Safety

IMPORTANT

The heat pad is fitted with a SAFETY SYSTEM. This sensor technology provides protection against overheating across the entire surface of the heat pad with an automatic switch-off in the event of a fault.

5.2 Initial use

IMPORTANT

Ensure that the heat pad will not bunch up or become folded during use.

It is possible to position the plug-in coupling at either the bottom left or bottom right of the heat pad. Please first check on which side the socket is closest. To change the position, open the zip fastening on the rear, remove the moulded foam part, push the plug-in coupling from the outside through the cover to the inside, and then guide the plug-in coupling back out through the other opening. Then re-insert the moulded foam part and close the zip fastening.

Use at home or at work

- Fasten the heat pad to the backrest of the chair by opening the two hook-and-loop fastening bands at the side (4) and connecting and closing them at the rear of the backrest.
- First, connect the mains part to the heating pad by inserting the plug-in coupling (5).
- Then insert the mains part (2) into the socket. The ON/OFF button with LED display (6) will flash briefly.


Use in cars


IMPORTANT

- Under no circumstances use the hook-and-loop fastening bands at the side of the heat pad to attach it to the car seat! Doing so would impair the function of the side airbag!
- Therefore, please close the side fastening bands at the rear of the heat pad, so that they do not interfere with anything and the hook-and-loop sections cannot damage the backrest. Once closed, they should lie flat against the rear of the heat pad.


First attach the car fastening straps (7) to the car seat. Place the car fastening strap on the seat surface so that the smoother side is facing upwards.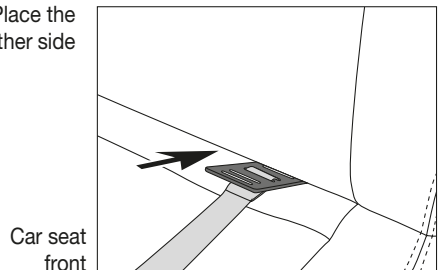


Push the rectangular securing catch through from the front to the back between the seat and backrest, and orient the securing catch vertically at the back.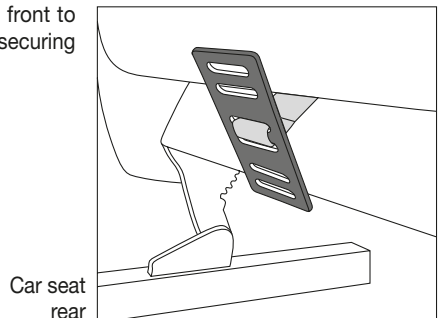

Then attach the clip to the headrest support. To do so, slide the two ends apart towards the sides, as shown in the diagram.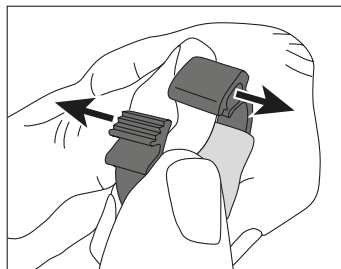

Then use this opening to guide the clip over the headrest support so that it can be closed again. Close the clip by pressing it together until it securely encloses the headrest support.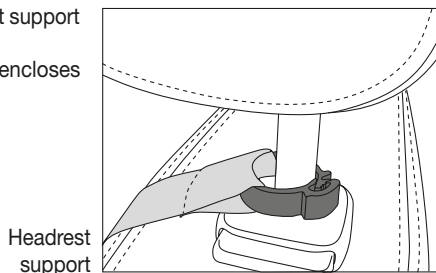

The car fastening straps must not cross one another, be entangled, or be mounted with the loop side facing the seat.


Now take the heat pad in your hand, open the covers of the 4 hook-and-loop points on the rear, hold the heat pad against your back with the curved side at the level of the lumbar vertebrae, and lean back against the backrest.

Then check the hook-and-loop connection. If the hook part does not lie fully on the loop part, you can now correct it, continuing until all 4 points are securely fastened.

You can change the position of the cushion at any time using the hook-and-loop fasteners on the rear of the heat pad.

Removal of car fastening straps from car seat

To open the clip, slide the two ends apart again towards the sides. Then push the rectangular securing catch through from the back to the front between the backrest and the seat.

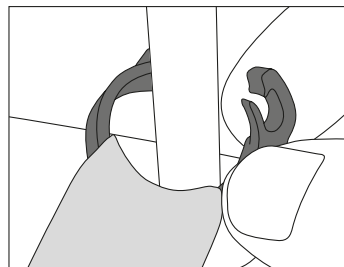5.3 Switching the device on

To switch on the heat pad, press and hold the ON/OFF button (6). The ON/OFF button will light up red. Temperature setting 3 – maximum heat – is active.

NOTE

Depending on the make, the 12-V socket in the vehicle may be activated at different times:

- permanently
- after opening the vehicle
- depending on the ignition or on whether the engine is running

IMPORTANT

If the 12-V socket in the car is permanently activated, we recommend always disconnecting the car adapter from the 12-V socket when leaving the vehicle.

5.4 Setting the temperature

To change the temperature, briefly press the ON/OFF button. The switching sequence is 1-2-3-1-2 ...

Setting 0: OFF

Setting 1: Minimum heat – lights up green

Setting 2: Medium heat – lights up yellow

Setting 3: Maximum heat – lights up red

NOTE

The fastest way to warm up the heat pad is to initially set the highest temperature setting.

NOTE

This heat pad has a fast heating function, which allows the heat pad to warm up quickly in the first 10 minutes.

WARNING

If the heat pad is being used over several hours, we recommend that you set the lowest temperature setting on the controller to avoid overheating the heated body part, which may lead to burns to the skin.

5.5 Automatic switch-off

This heat pad is equipped with an automatic switch-off function. This turns off the heat supply approx. 90 minutes after the initial use of the heat pad. The ON/OFF button is then no longer illuminated. You can switch the heat pad on again by pressing the ON/OFF button.

5.6 Switching the device off

To switch off the heat pad, press and hold the ON/OFF button. The ON/OFF button is then no longer illuminated.

NOTE

If the heat pad is not being used, press and hold the ON/OFF button to switch off the heat pad, and pull the mains part from the socket or the car adapter from the 12-V socket in the car.

6. Cleaning and maintenance

WARNING

Before performing cleaning work, always first pull the mains part from the socket/the car adapter from the 12-V socket in the car. Then disconnect the mains part/car adapter from the heat pad by unplugging the plug-in coupling. Otherwise there is a risk of electric shock or damage to the heat pad or adapters.

IMPORTANT

The mains part and car adapter must never come into contact with water or other liquids. Otherwise, this could damage them.

- Use a dry, lint-free cloth to clean the mains part and car adapter. Do not use any chemical or abrasive cleaning agents.
- Small marks on the heat pad can be removed with a damp cloth and, if necessary, with a little liquid detergent for delicate laundry.

IMPORTANT

Please note that the heat pad must not be chemically cleaned, wrung out, tumble dried, put through a mangle or ironed.

- This heat pad is machine-washable. To do so, open the zip fastening on the rear and remove the moulded foam part. Removing the moulded foam part means the heat pad can dry faster.
- Place the heat pad in a pillowcase and close it. The pillowcase will reduce the mechanical strain on the heat pad while it is being washed.
- Set the washing machine to a particularly gentle wash cycle at 30°C (wool cycle). Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

IMPORTANT

Please note that frequent washing of the heat pad has a negative effect on the product. The heat pad should therefore be washed in a washing machine a maximum of 10 times during its life.

Remove the heat pad from the pillowcase immediately after it has been washed. Reshape the heat pad to its original dimensions while it is still damp and spread it out flat on a clothes horse to dry.

IMPORTANT

Do not use pegs or similar items to attach the heat pad to the clothes horse. Otherwise the heat pad may be damaged.

Once the heat pad is completely dry, you can re-insert the moulded foam part and close the zip fastening.

WARNING

Do not reconnect the mains part or the car adapter to the heat pad until the plug-in coupling and heat pad are completely dry. Otherwise the heat pad may be damaged.

WARNING

Never switch the heat pad on to dry it! Otherwise there is a risk of electric shock.

7. Storage

If you do not plan to use the heat pad for a long period of time, we recommend that you store it in the original packaging. For this purpose, disconnect the heat pad from the mains part/car adapter by unplugging the plug-in coupling.

IMPORTANT

- Please allow the heat pad to cool down before storing. Otherwise the heat pad may be damaged.
- To avoid sharp folds in the heat pad, do not place any objects on top of it while it is being stored.

8. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

9. What if there are problems?

Problem	Solution
The ON/OFF button is not illuminated while - the mains part is properly connected to the heat pad - the mains plug is connected to a working socket - the ON/OFF button was pressed and held to switch on the device	Send the heat pad and the mains part for servicing.

10. Technical data

See the type plate label on the heat pad.

Technical data for mains part

Model identifier:	CZH024120200BSWM (BS-plug)
Input voltage:	100 V-240 V
Input AC frequency	50 Hz/60 Hz
Output voltage:	12.0 V
Output current:	2.0 A
Output power:	24.0 W
Average efficiency during operation:	86.5%
Efficiency at a low load:	85.9%
No-load power consumption:	0.05 W
Name/trademark of the manufacturer:	Shenzhen Chanzeho Technology Co. , Ltd.

11. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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