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that you do not damage or displace the seal.

Empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries. The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

This battery contains: Pb, Hg.

#### Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- **⚠ Choking hazard!** Small children may swallow and choke on batteries or the battery compartment. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- **⚠ Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

#### Cleaning the device

Clean the basal thermometer before and after each use with a soft cloth and isopropyl alcohol diluted with water or cold soapy water. The device can be immersed in water or a disinfectant solution to clean it. The maximum immersion time is 30 minutes. Do not sterilise the device by boiling, with gas or with an autoclave.

#### Storage and disposal

Please store the thermometer in the original protective cover when it is not in use. The accuracy of this thermometer has been carefully checked and it has been developed with a long service life in mind. If the device is used for commercial medical purposes, it must be regularly tested for accuracy by appropriate means. Precise instructions for checking accuracy may be requested from the service address.

For environmental reasons, do not dispose of the device in the household waste at the end of its service life. Dispose of the device at a suitable local collection or recycling point in your country.

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#### Do you have any questions about the Ovy app?

The Ovy team is on hand to help in such cases and you are welcome to e-mail us at hello@ovyapp.com. The Ovy App has a contact function in your profile, which you can also use to send us your questions and feedback directly. The Ovy website and the information sections in the app already offer some answers and diverse background information about your cycle.

Website: www.ovyapp.com

Contact: hello@ovyapp.com

Facebook: Ovy

Instagram: @ovyapp

#### Warranty / service

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international". In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

1. The warranty period for BEURER products is either 5 years or- if longer- the country specific warranty period from date of Purchase.
- In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages because of:
  - a. improper treatment, e.g. nonobservance of the user instructions.
  - b. repairs or tampering by the customer or unauthorised third parties.
  - c. transport from the manufacturer to the consumer or during transport to the service centre.
  - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

Subject to error and change

Site Web : www.ovyapp.com

Contact : hello@ovyapp.com

Facebook : Ovy